OMNI

ONE LIFE MEANS EVERYTHING

Multi Organ Support Therapy (MOST)

Fluid overload
 Pulmonary Edema
 Cerebral Edema
 Transplant

SCUF
 CVVH

Azotemia
 Acidosis
 Uremia
 Hypervolemia
 Anuria/Oliguria
 Hyperkalemia
 Transplantation

SCUF
 CVVH
 CVVHD
 CVVHDF
 Plasmapheresis

Fluid overload
 Pulmonary Edema
 Transplantation

SCUF
 CVVH
 Plasmapheresis
 ECCO₂R

SIRS/Sepsis
 Transplantation

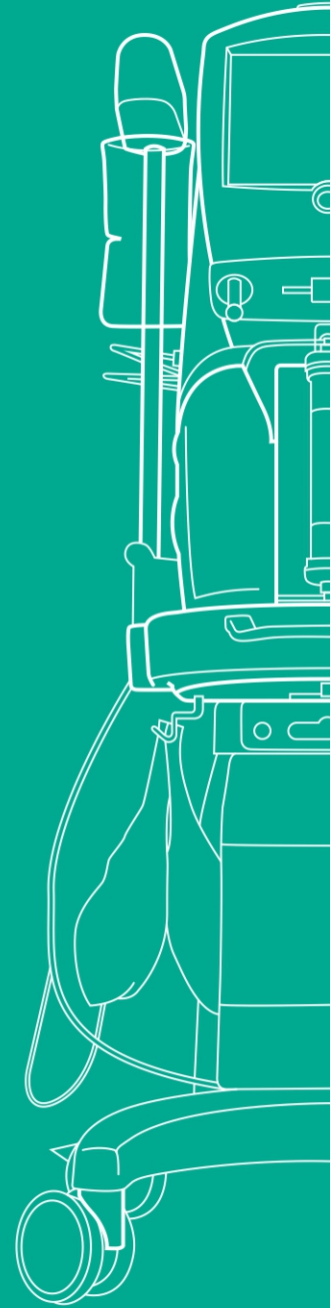
Plasmapheresis
 DFPP

SCUF: Slow Continuous Ultrafiltration
 CVVH: Continuous Veno-venous Hemofiltration

CVVHD: Continuous Veno-venous Hemodialysis
 CVVHDF: Continuous Veno-venous Hemodiafiltration

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CRRT Therapies

SCUF: Slow Continuous Ultrafiltration	✓
CVVH: Continuous Veno-venous Hemofiltration	✓
CVVHD: Continuous Veno-venous Hemodialysis	✓
CVVHDF: Continuous Veno-venous Hemodiafiltration	✓
Pre - Post - Pre/Post - Post/Post	✓


Anticoagulation Modes

Heparin	✓
Citrate	✓

Adjuvant Therapies

HP: Hemoperfusion	✓
TPE: Therapeutic Plasma Exchange	✓
ECCO ₂ R: Extra Corporeal CO ₂ Removal	✓

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
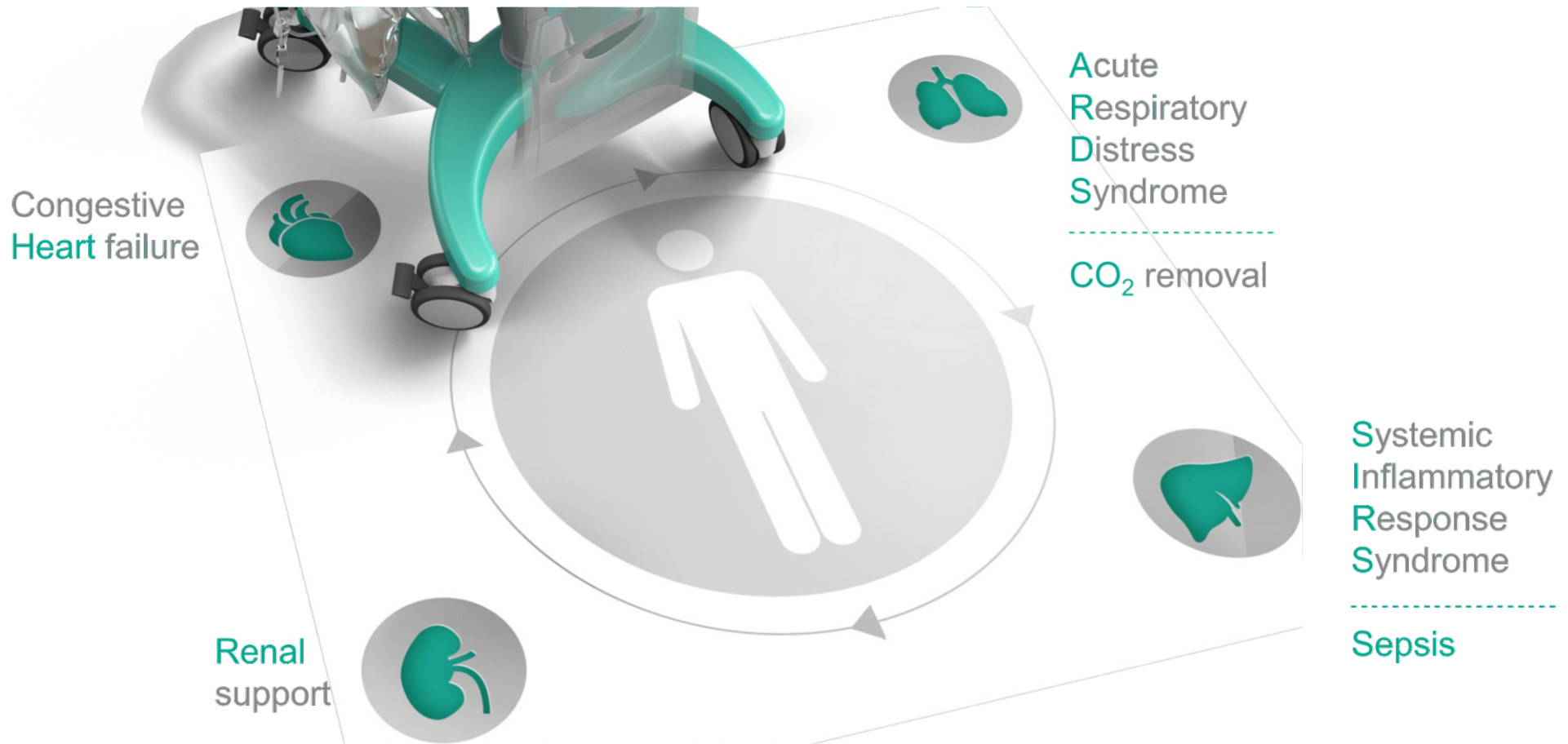
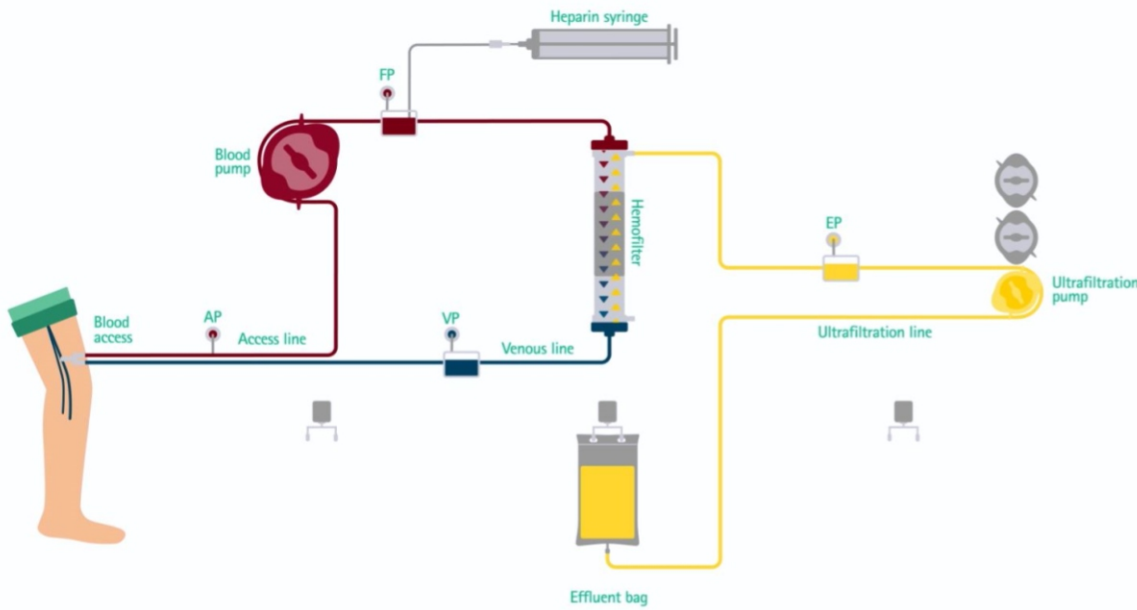
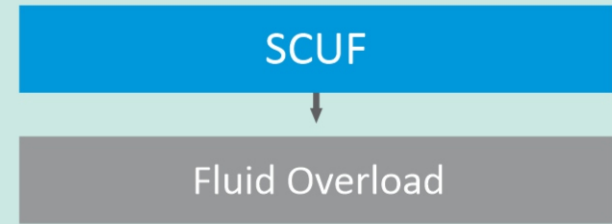
SCUF: Slow Continuous Ultrafiltration

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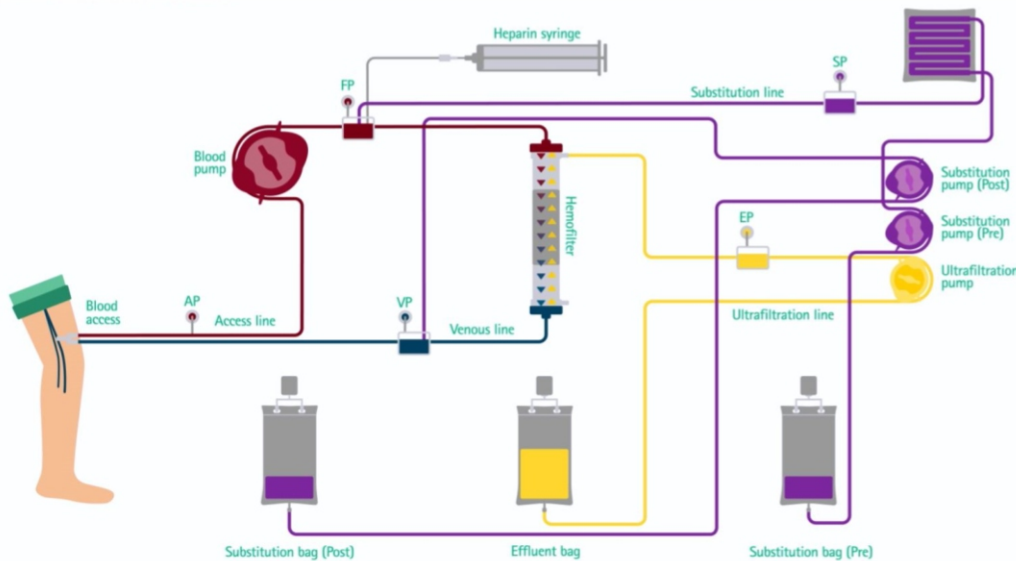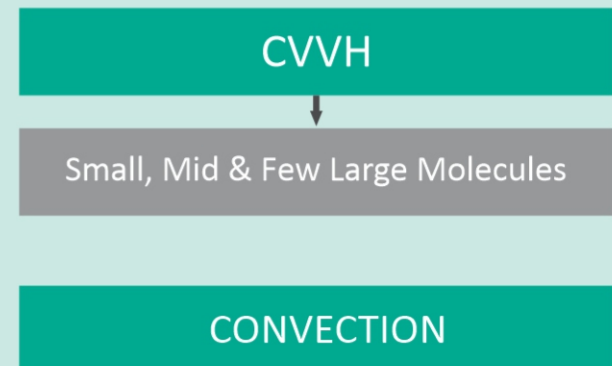CVVH: Continuous Venovenous Hemofiltration

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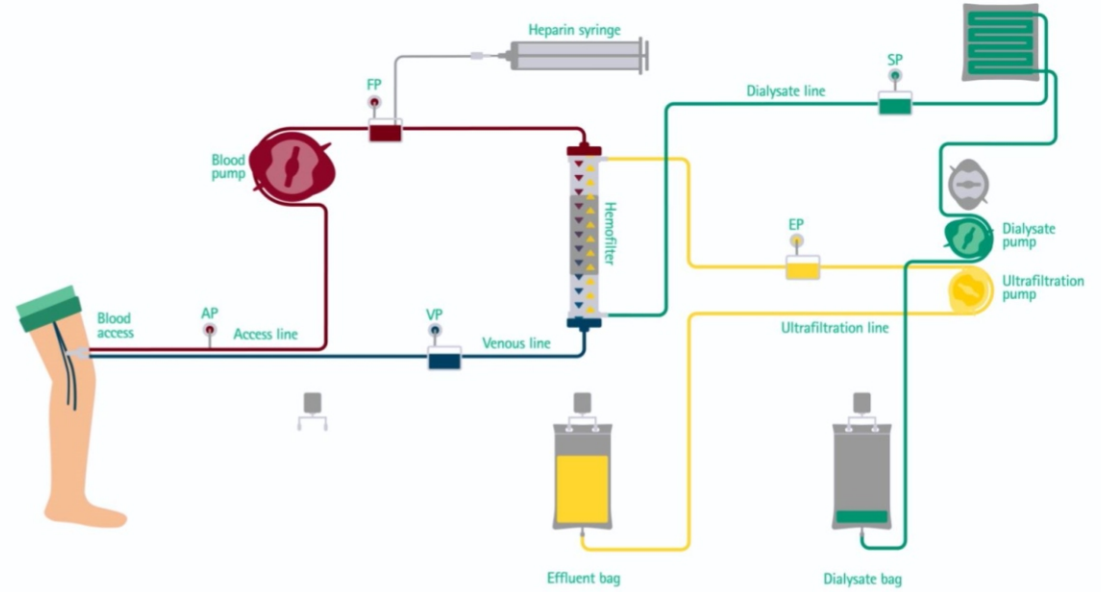
Small & Mid Molecules

DIFFUSION

CVVHD: Continuous Veno-Venous Hemodialysis

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
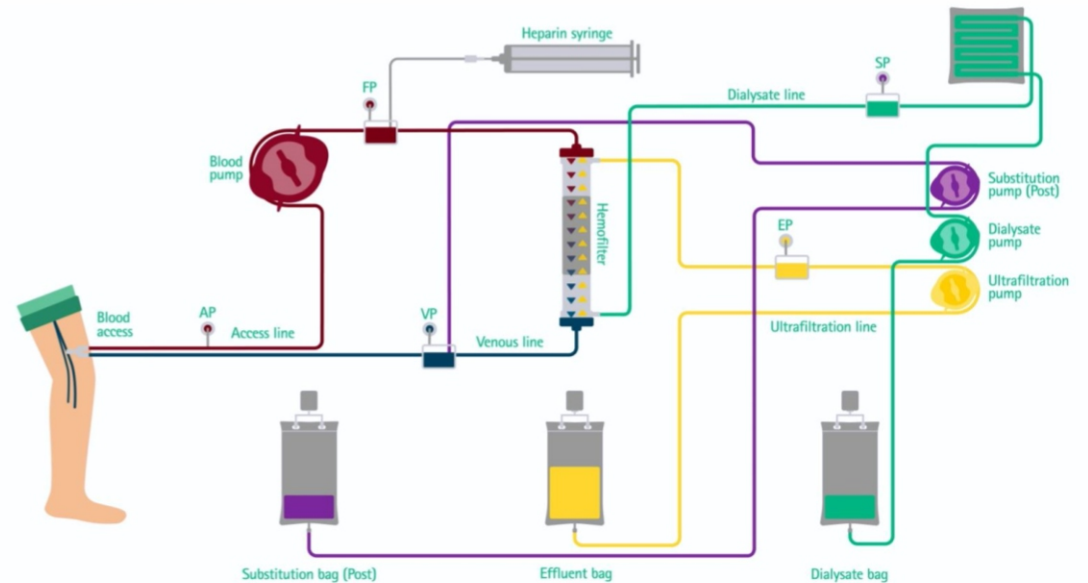
DIFFUSION

CONVECTION

CVVHDF: Continuous Veno-Venous Hemodiafiltration

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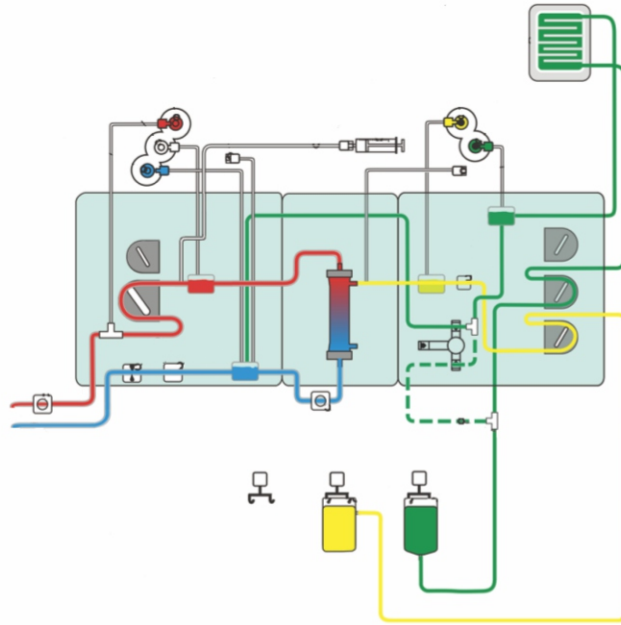ECMO Cannula
18 F \approx 6 mm

Dialysis Cannula
16 G \approx 1.6 mm

Comparison of Extracorporeal Therapies

Extracorporeal Support Technics


	CRRT Continuous Renal Replacement Therapy	ECCO ₂ R Extra Corporeal CO ₂ Removal	ECMO Extra Corporeal Membrane Oxygenation		
Blood Flow L/min	0.2-0.3	0.3-0.5	1.0-2.5	2.0-5.0	5.0
Vascular Access	V-V	V-V	A-V	V-V	A-V
Catheter \varnothing F	12 Double Lumen	14 Double Lumen	V 13 A 15	Outlet 15-21 Inlet 19-24	V 19-24 A 16
Priming Volume ml	140-160	300	350	500	500
Heparin IU/min	4-12	4-18	3.5-10	10-20	10-20
Membrane Surf. m ²	---	1.3	1.3	1.8	1.8
CO ₂ Extraction % of Baseline	---	25	50	>50	>50
O ₂ Transfer ml/min	---	10	20-60	140-340	340

Delivery of O₂ - Clearance of CO₂

Membrane Types:

- Polypropylene
- Polymethylpentene coated by Phosphorylcholine
- Hollow fiber / spiral coil type

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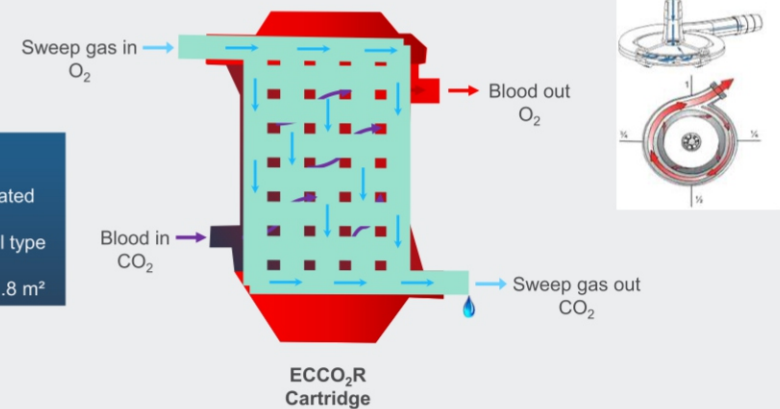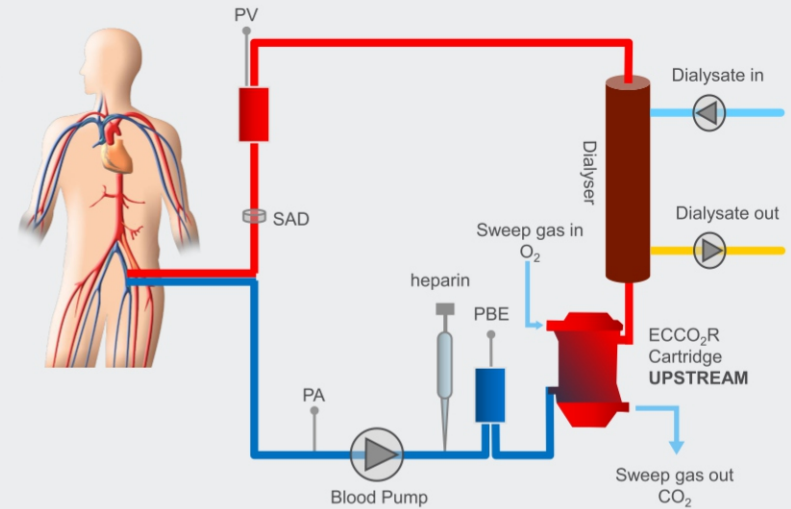
OMNiset® ECCO₂R ویژگی

- Preassembled OMNiset with 1.6 m² Hemofilter
- Oxygenator (Eurosets) 1.8m² polymethyl pentene membrane coated by phosphorylcholine
- Blood Flow: up to 500 ml/min
- Position of Oxygenator (pre Filter)
- Anticoagulation: Heparin
- Max treatment time 72h
- CVVHD+CVVHDF+CVVH pre/post


ECCO₂R + CRRT

- Indications:**
- ICU patients with hypercapnia and AKI
 - Protective Ventilation
 - Prevent Intubation
 - Enhance NIV
 - Early Extubation
 - Bridge to lung Transplantation

Low Flow ECCO₂R
300-500 ml/min



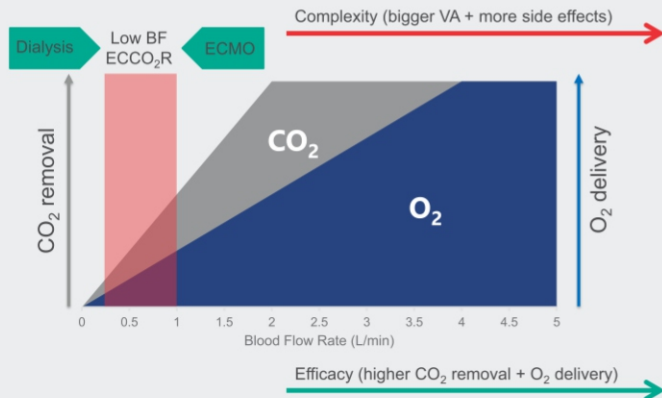
Gas exchange in extracorporeal circulation

Clinical Position of ECCO₂R in ARDS

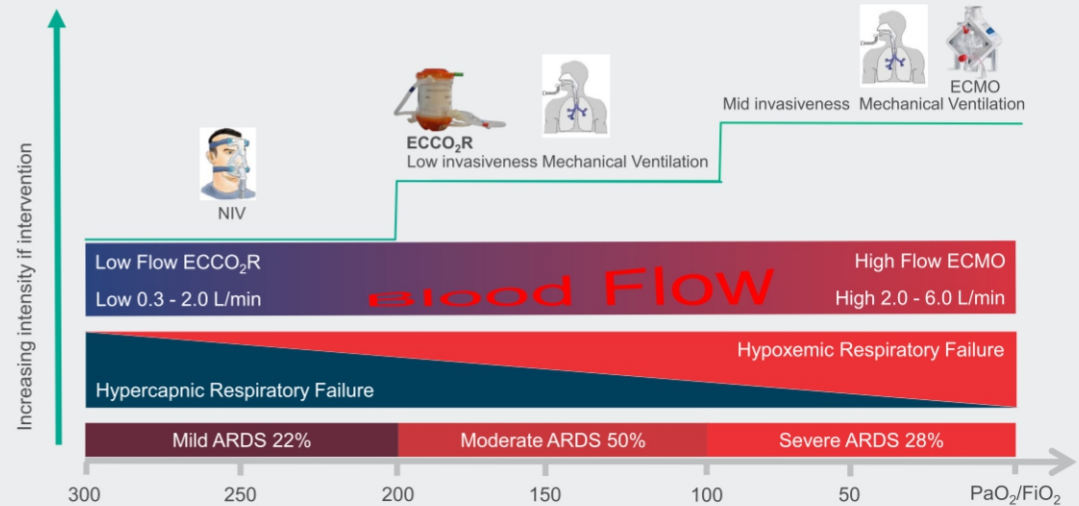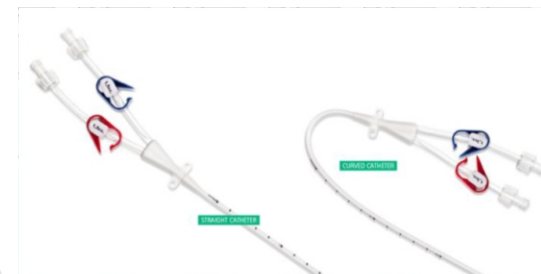
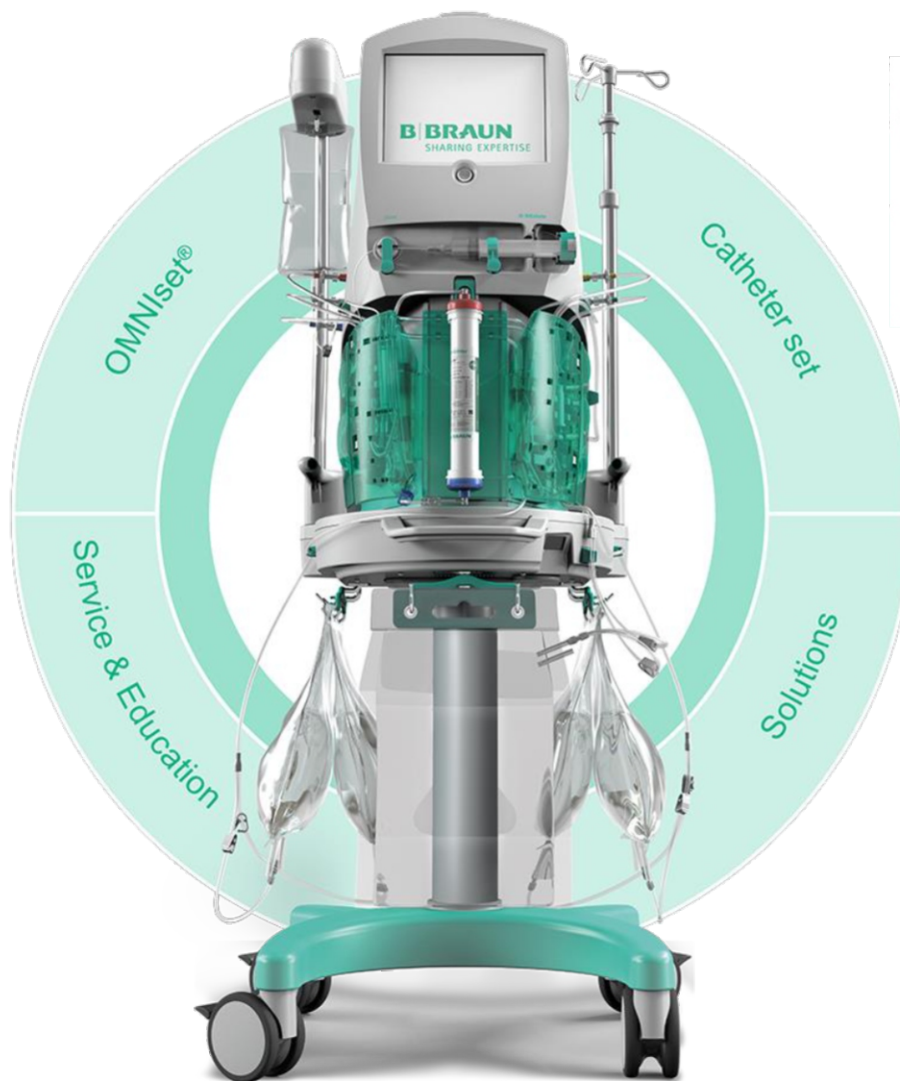
Berlin definition of ARDS

OMNI

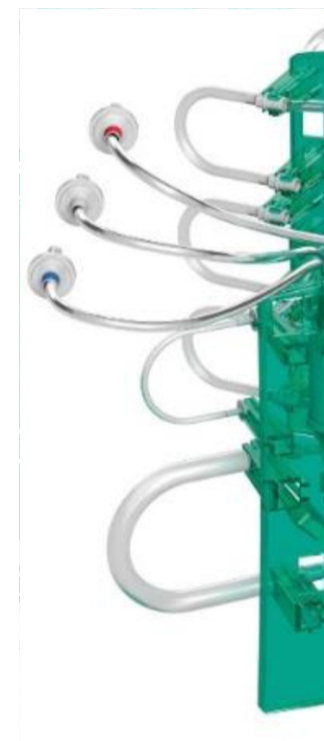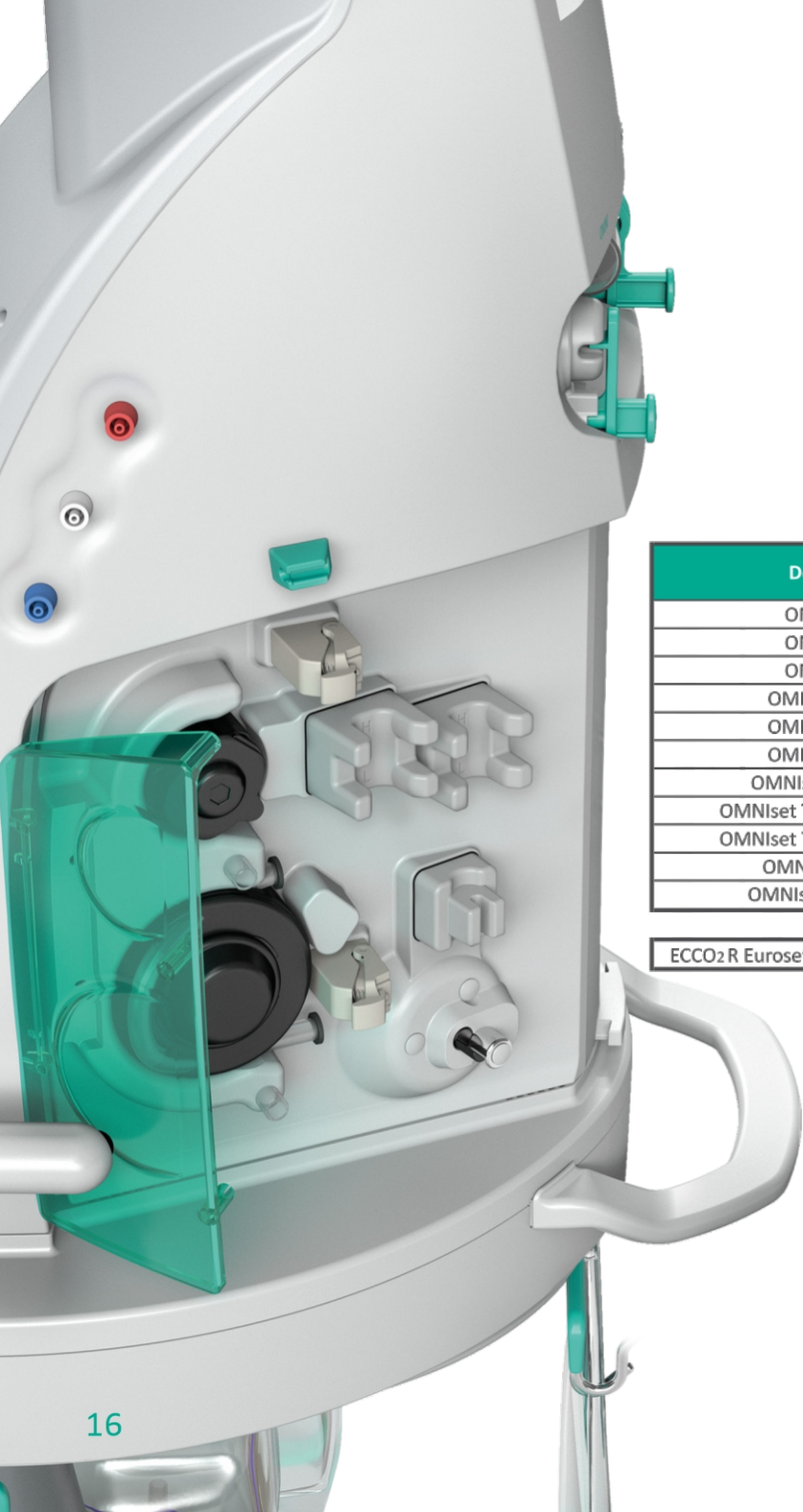Description	CVVH Heparin	CVVHD Heparin	CVVHD Citrate	CVVHDF Heparin	CVVHDF Citrate	TPE	SCUF	CO2 Removal
OMNiset 0.8 m ²	✓	✓		✓			✓	
OMNiset 1.2 m ²	✓	✓		✓			✓	
OMNiset 1.6 m ²	✓	✓		✓			✓	
OMNiset Pro 0.8 m ²		✓	✓	✓	✓		✓	
OMNiset Pro 1.2 m ²		✓	✓	✓	✓		✓	
OMNiset Pro 1.6 m ²		✓	✓	✓	✓		✓	
OMNiset L CRRT 1.6 m ²		✓	✓	✓	✓		✓	
OMNiset TPE incl FILTER 0.5 m ²						✓		
OMNiset TPE incl FILTER 0.7 m ²						✓		
OMNiset Plus 1.6 m ²		✓	✓	✓				
OMNiset ECCO2R 1.6	✓	✓		✓				✓

ECCO2R Eurosets Oxygenator 1.81 m²

OMNI Fluid | Replacement, Dialysate and Citrate solution

Duosol (Drugs) or Sterile Bicarbonate Solution (MD)

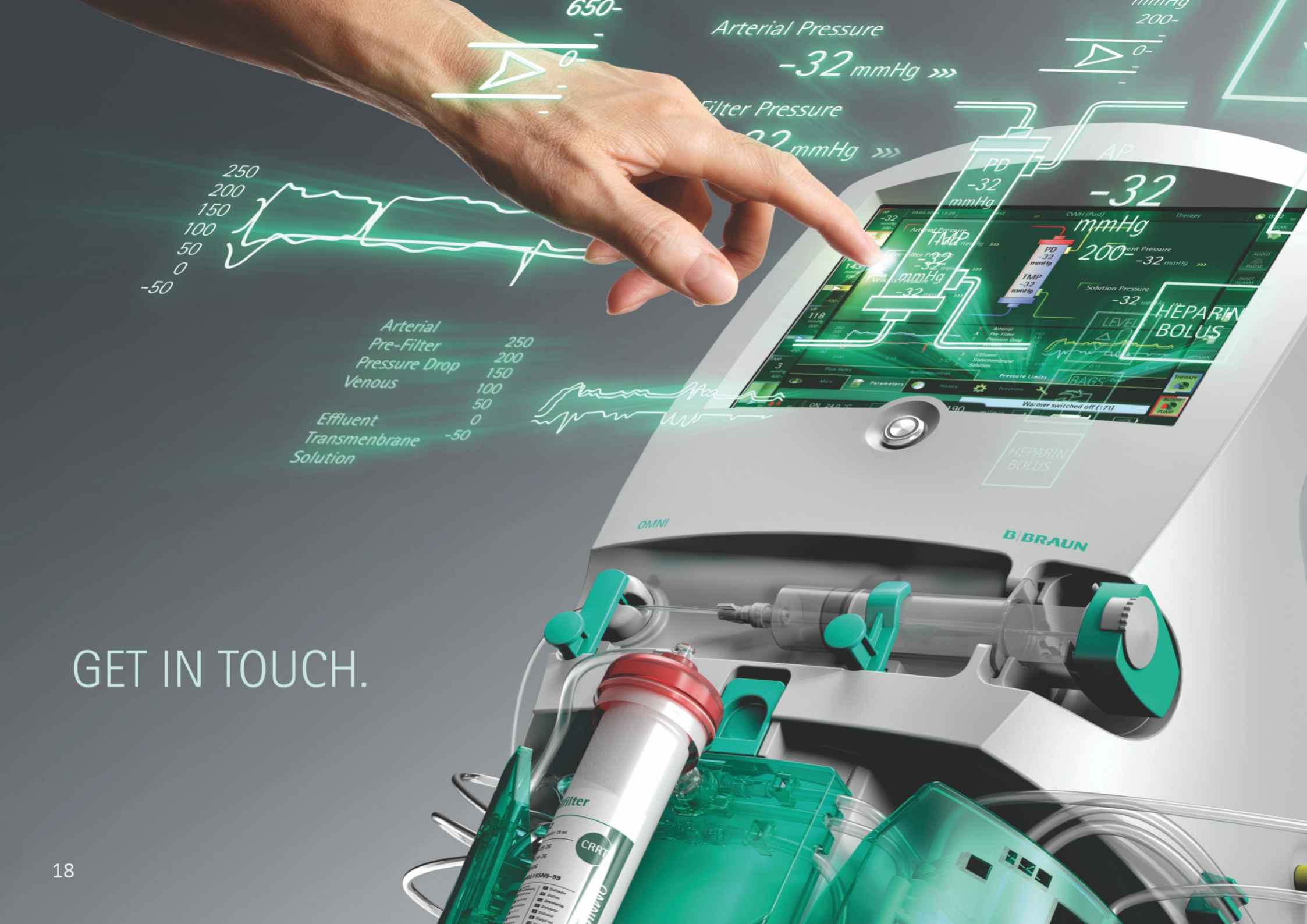
Bicarbonate buffered concentration for replacement of Fluid (via Pre or Post dilution)

Calcium Free Solution

The calcium-free bicarbonate solution which used as dialysate for CVVHD/HDF Regional Citrate anticoagulation

Citrate 4% Solution

Trisodium citrate 4% solution for the Regional Citrate Anticoagulation

GET IN TOUCH.

Full spectrum of CRRT modalities

- Renal support (AKI)
- Fluid Overload
- Individualized therapy approach

Full spectrum of therapy and anticoagulation modes

- SCUF, CVVH, CVVHD, CVVHDF, pre - post - pre/post - post/post
- Heparin/ citrate anticoagulation modes
- TPE**
- Low blood volume**

Full flexibility during therapy

- Continuous monitoring of the patient's condition
- Renal dose follow-up on the screen linked to fluid-balance management
- Easy to adjust within any therapy
- Easy to change the dilution (pre/post)
- Easy to change the anticoagulation mode*
- Easy to change to another therapy form with the same set

Most accurate fluid-balance management

- Renal dose adjustment
- Four fluid pumps
- One syringe pump for heparin & calcium modes
- Three-way clamp for full therapy flexibility
- Automatic blood-flow reduction (HSP - HomeoSAFE Premium)
- Automatic fluid-exchange decrease (HSP)
- Automatic air removal via chamber regulation
- Reduced bag changes
- Ceramic fluid heater

Unique usability

- Fully pre-assembled kit with integrated OMNifilter
- Lower blood volume thanks to fully pre-assembled set
- Short prep times
 - Easy setup (loading, connecting, hanging bags on the load cells)
 - Fast priming
 - Integrated warmer preparation
- Renal dose adjustment

Intuitive user interface

- 12" touchscreen
 - Graphic user interface
 - Intuitive video tutorials and help screens
 - Monitoring renal dose and patient condition
- Staff call
- 360° traffic status light

Cutting-edge design

- One of the sleekest and lightest device in the market
- Easy to move around the ICU
- One of the smallest footprint
- Easy-to-clean surfaces

Advanced safety features

- Dual-channel safety system
- Barcode scanner (avoids setup errors)
- Backup battery, 30 min. (easy and safe intrahospital transport)
- Data recording, patient data management system (PDMS)
- Treatment history of the last 10 patients
- Three-way clamps for therapy/dilution changes
- Adult/pediatric options

360° portfolio

Comprehensive system consisting of:

- OMNI
- OMNiset hemo (fully preconnected set incl. hemofilter with four different surface choices)
- OMNiset plasma** (fully preconnected set incl. plasmafilter with three different surface choices)
- Catheter Set (curved and straight options)
- Duosol (three different bicarbonate formulas)
- Calcium-Free Bicarbonate Solution
- Citrasol

Expert support

- Dedicated team for superior support and product training
- Specific workshops and symposia
- B. Braun Avitum ABP Academy
- Service contracts
 - * the change of anticoagulation mode is allowed only between therapies, not during the therapy
 - * future option

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